

Family Law: Client Info Packet

Effortless Alcohol Monitoring. Finally.



Basic Information, How to Schedule Tests, Testing Tips, Pricing

Start your free trial at monitoring.bactrack.com



Video-verified proof of sobriety.



BACtrack View is an app-based, remote alcohol monitoring service that helps users prove sobriety - starting at only \$79.99/month.



Police-Grade Accuracy

Each user is given a discreet BACtrack Mobile smartphone breathalyzer, which uses a police-grade fuel cell sensor to deliver reliable and accurate results. Every test, every time.



Automated Accountability

SMS messages prompt users when it's time to take a breathalyzer test, and the video-verified test results are instantly transmitted to the app for monitors to review.



Court-Ready Records

It's easy to download a PDF report to submit in court. All reports neatly display the time, date, and recorded BAC of each test.



How to Schedule Tests

& make BACtrack View work for any visitation arrangement

Scheduling breathalyzer tests in the BACtrack View app is easy and totally flexible - you can adapt it to any parenting time arrangement, no matter how complex.



Randomized




Customized



Repeating




On-Demand

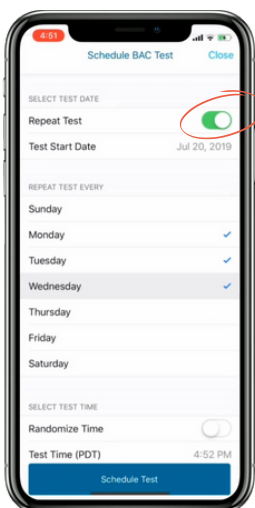

Request an on-demand test... or set a schedule.

If you'd like to schedule a single, non-repeating test, enter the test date and time. You're ready to go.


To create a schedule: Hit the "Repeat Test" button (circled). Set the date you'd like testing to begin, and which days of the week tests will occur.

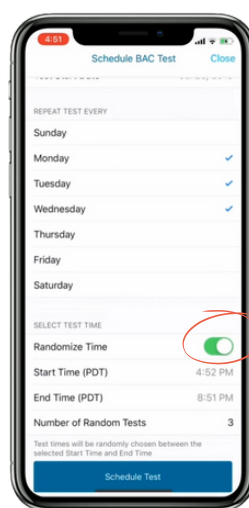

Define or randomize testing times.

If you'd like the Tester to be prompted at specific hour, just enter the "Test Time" and you're done.

Alternatively, you can have tests occur at random times by hitting the "Randomize Time" button (circled). Select the time frame during which your Tester will be prompted, and the number of random tests your Tester will take during that period.

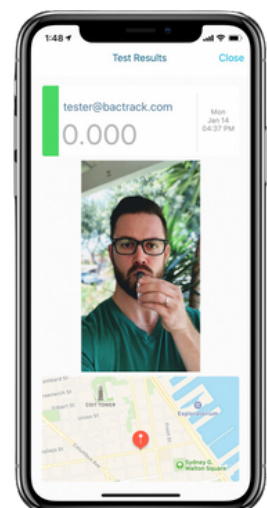

Get real-time results, straight to your phone

All you have to do now is wait for the results.

Video and location-verified BAC results are captured and saved via the app, and automatically shared with monitors via text or email notifications.



Sample Testing Schedules

- Randomized vs. Non-Randomized
- Scheduling for Custody
- Sample Schedule

Sample #1: Non-Randomized, Repeating

This may be the ideal option if you want testing to occur only at a very specific time - when parenting time begins, when a child is picked up, etc.

Let's say you want to test every Monday, Thursday, and Saturday at 9:00 am. Here is what that schedule will look like (below):

	✓ Monday	Tuesday	Wednesday	✓ Thursday	Friday	✓ Saturday	Sunday
7:00 am							
8:00 am							
9:00 am	test			test		test	
10:00 am							
11:00 am							
12:00 pm							
1:00 pm							

Sample #2: Randomized, Repeating


Randomized testing can be ideal to ensure a co-parent remains sober throughout a time frame - i.e. during visitation - without them anticipating the exact hour/minute tests will occur.

Let's say you want to randomly test a co-parent every Monday, Thursday, and Saturday. You want them to receive 3 randomly-timed tests between 8:00 am - 1:00 pm. Here is what that schedule will look like (below):

	✓ Monday	Tuesday	Wednesday	✓ Thursday	Friday	✓ Saturday	Sunday
7:00 am							
8:00 am	Random test schedule: Test 1: ? Test 2: ? Test 3: ?			Random test schedule: Test 1: ? Test 2: ? Test 3: ?		Random test schedule: Test 1: ? Test 2: ? Test 3: ?	
9:00 am							
10:00 am							
11:00 am							
12:00 pm							
1:00 pm							



Pricing

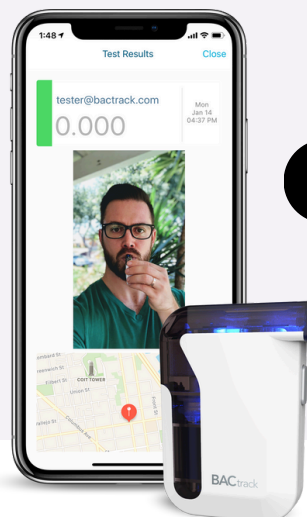

Available on iOS
+ Android

	Basic	Plus <small>POPULAR</small>	Pro
	\$79.99 / month	\$99.99 / month	\$129.99 / month
FREE* pro-grade, smartphone breathalyzer	✓	✓	✓
Tests per month	30	Unlimited	Unlimited
PDF Reports	None	1 per month	Unlimited
Connections	1	3	Unlimited
Monitor Notifications	Text	Text	Text + Email
Customer Service Level	Email	Phone + Email	Dedicated Account Rep

- *Breathalyzers are free for you to use as long as you have a paid monthly subscription. If you complete 3 months on your plan, the device is yours to keep, free of charge. If you decide to end your subscription before 3 months, you can either return the device to us or purchase it outright at a discount. Free trials are for new customers only.
- If a Tester reaches their "Tests per month" limit before the month has ended, they will not be able to take more tests unless they upgrade to a higher plan.
- PDF reports are customizable by time period and include the Tester name, test date and time, and BAC test result.
- A connection is a Monitor or Accountability partner.
- Monitors receive text or email notifications (or both) when a test has been taken or missed.



Get started with
your **14-day**
free trial.

monitoring.backtrack.com
415-693-9194
view@backtrack.com



Gary, real BACtrack View user

Click here to start your
14-day free trial.

BACtrack® | VIEW

