



# Family Law: Professional Info Packet

Effortless Alcohol Monitoring. Finally.

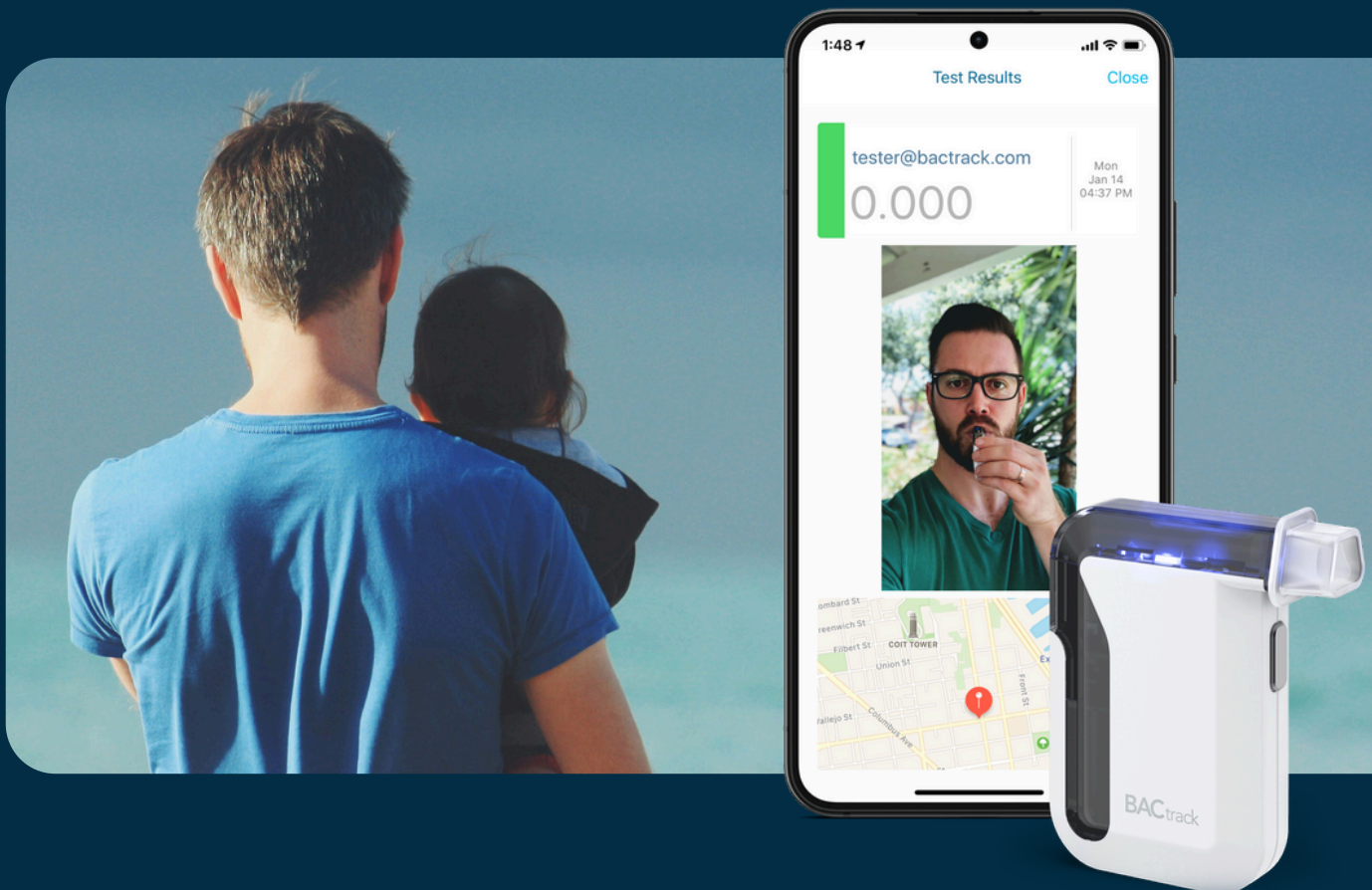

Alcohol Monitoring Cheat Sheet, Reporting, About BACtrack View



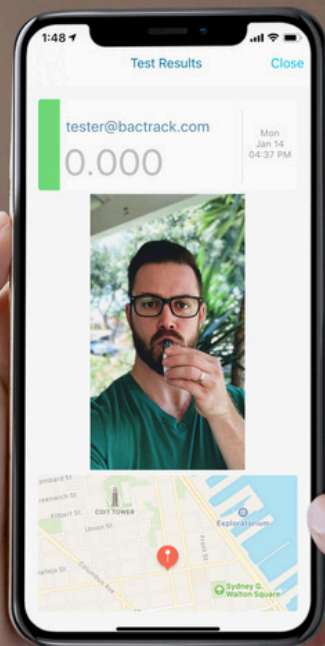

## Get video-verified proof of a parent's sobriety

- ✓ Access all video and location-verified breathalyzer test results in the app
- ✓ Set random, recurring or custom testing schedules for individual users
- ✓ Monitors receive real-time email or SMS notifications for failed or missed tests

**BACtrack View is an app-based remote alcohol monitoring service that helps users prove sobriety - starting at only \$79.99/month.**

### Police-Grade Accuracy

Each user is given a BACtrack Mobile smartphone breathalyzer, which uses a police-grade fuel cell sensor to deliver reliable and accurate results. Every test, every time.

### Automated Accountability

SMS messages prompt users when it's time to take a breathalyzer test, and the video-verified test results are instantly transmitted to the app for monitors to review.



### Court-Ready Records

Parents and counsel can easily download a PDF report to submit in court. All reports neatly display the time, date, and recorded BAC of each test.


# Reporting

Presenting evidence via  
a BACtrack View PDF  
Report


What makes BACtrack View ideal for custody cases is the ability to download reports of a parent's BAC results to share and present in court. And it only takes seconds.

This option is not available for Basic plan users. Plus plan users can download 1 PDF report per month, while Pro plan users have unlimited PDF report downloads.


Here's how it works:

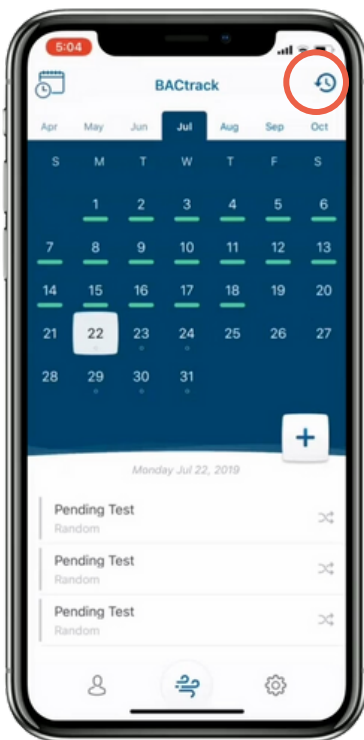

Hit the clock button in  
the corner of the  
calendar (circled).

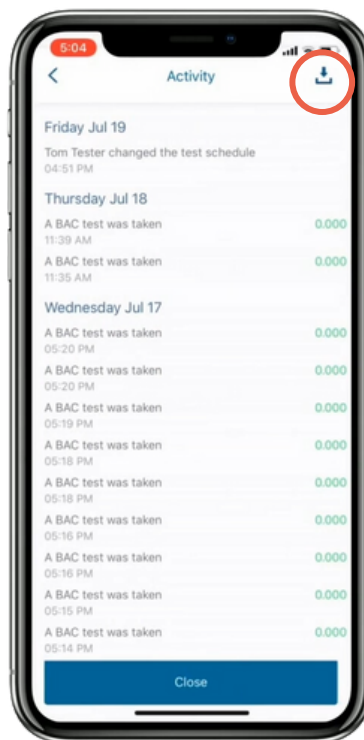

Press the download  
button in the corner  
(circled).

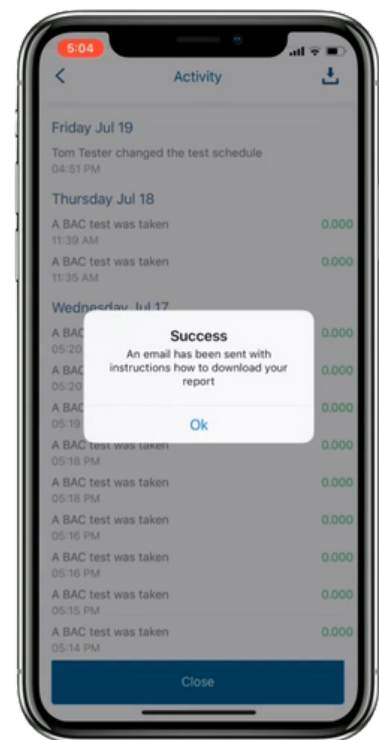

You'll receive an  
email with further  
instructions.



# Reporting

Presenting evidence via  
a BACtrack View PDF  
Report

**Detail Report for Kevin Example**  
February 4 – 10, 2024

**BACtrack | VIEW**

**Client**  
Kevin Example  
123 Main St.  
Anytown, CA 9423  
ew@email.com  
(123) 456-7890

**Monitors**  
John Doe: name@email.com  
Jack Smith: name@email.com  
Sally Doe: name@email.com

**BAC Goal: 0.00**  
Report Run: March 7, 2024  
Time Zone: Pacific Standard Time

**Summary**

Total Results	15
Compliant Tests	10
Missed Tests	2
Non-compliant Tests	3

**Comments**

**Detail**

DATE	TIME	INITIATED BY	ACTIVITY	RESULT
February 10, 2024	7:36pm	Scheduled	BAC test taken	BAC level: 0.082
February 10, 2024	6:00pm	Scheduled	Scheduled test was missed	-
February 9, 2024	10:23am	Self	BAC test taken	BAC level: 0.000
February 9, 2024	10:23am	Self	BAC test taken	BAC level: 0.000
February 8, 2024	10:54pm	Self	BAC test taken	BAC level: 0.021
February 8, 2024	12:02pm	Scheduled	BAC test taken	BAC level: 0.000
February 7, 2024	8:45pm	Self	BAC test taken	BAC level: 0.000
February 7, 2024	7:00pm	Scheduled	Scheduled test was missed	-
February 7, 2024	6:08am	Self	BAC test taken	BAC level: 0.000
February 6, 2024	11:17pm	Self	BAC test taken	BAC level: 0.000
February 6, 2024	5:23pm	Self	BAC test taken	BAC level: 0.000
February 5, 2024	9:03pm	Scheduled	BAC test taken	BAC level: 0.000
February 5, 2024	6:42pm	Self	BAC test taken	BAC level: 0.061
February 4, 2024	3:23pm	Self	BAC test taken	BAC level: 0.000
February 4, 2024	11:01am	Scheduled	BAC test taken	BAC level: 0.000

1

Every PDF report displays the Tester's name, along with the time period the report covers.

2

The Summary tallies the total results, and distinguishes between non-compliant and missed tests.

3

Every test result is listed out – indicating BAC level, how it was initiated, date, and time – and color-coded.





# Alcohol Monitoring 411



Brooke, real BACtrack View user

## Overview

Remote alcohol monitoring provides a black-and-white measure of a parent's sobriety at any given moment in time.

BACtrack View prompts co-parents to take breathalyzer tests using an app-connected device. They are randomly tested within a specified time period, on a schedule, or both to prove they have not consumed alcohol. Designated parties will receive real-time, video-verified results of each test.

### Terms to Know

- **Tester** - the parent whose BAC level you wish to track.
- **Monitor** - the main party reviewing the Tester's BAC results. This person sets testing schedules, can request on-demand tests, and receives BAC test results in real-time.
- **Accountability Partner** - a party reviewing the Tester's BAC results. This person receives BAC results in real-time but cannot set testing schedules or request on-demand tests.
- **Connection** - a Monitor or Accountability Partner.

### Key Considerations

Before you get started with any alcohol monitoring service, there are key factors to consider.

#### Affordability

Affordable alcohol monitoring - without hidden fees or long-term contracts - is far easier to maintain.

#### Accuracy

How accurate are the results? Are they stamped with key information - such as test dates and times?

#### Method of proof

As proof of identity: Does a service only capture still photos... or record a video of the tester actually testing?

#### Customization

Is the scheduling flexible to adapt to complex parenting time schedules? Can you set up scheduled AND random tests?

#### Ease of use


Is the breathalyzer tiny and discreet enough to be used anywhere? Are notifications automated/real-time?

### Questions to Ask

- **Who will pay for alcohol monitoring?**
  - Either a Monitor or a Tester can submit monthly payments to BACtrack View, but not both.
- **What happens in the event of a missed or positive (non-compliant) test?**
  - What are the consequences and procedures?
- **How often will BAC reports be submitted to court?**
  - This will determine which plan you select.
- **Who will become the Monitor on the account?**
- **Who else will receive results?**
  - Will a co-parent or counsel become an Accountability Partner?
- **Which testing schedule will best ensure the child's safety?**
  - For example: During visitation, a pickup, etc.

- **Which testing schedule will best demonstrate a parent's commitment to sobriety?**
- **Will tests be scheduled, randomly-timed, or both?**
  - Should the Tester be able to anticipate the exact time of a test or not?
- **How many tests will the Tester submit per month?**
  - This will determine which plan you select.

# About BACtrack View

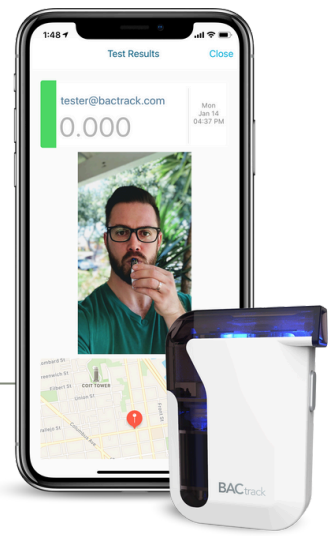

Founded in 2001, BACtrack is a pioneer in breathalyzers.

In 2004, it was the first company to obtain FDA clearance to manufacture and market breathalyzers for consumer use in the United States.

Since then, BACtrack has grown from its San Francisco roots into a global leader in the alcohol testing and monitoring space - providing some of the most accurate and innovative breath alcohol testing devices and software on the market.

**Founded:** 2001

**Headquarters:** San Francisco, CA

**Other Locations:** San Diego, CA - Philadelphia, PA - Austin, TX - Buffalo, NY

## BACtrack View is Court Approved ✓


BACtrack View is being used in courts nationwide by satisfied customers for their court appointed remote alcohol monitoring requirements.

View a sample of our current and past partners below.

- Washoe County, NV
- Warrick County, IN
- Bayfield County, WI
- Denton County, TX
- Lane County, OR
- Brown County, WI
- Mahoning County, OH
- Clayton County, GA
- Benton County, AR
- Kent County, MI
- Killeen County, TX
- San Diego County, CA
- Atascosa County, TX
- Calhoun County, MI
- Bell County, TX
- Clark County, NV
- Fresno County, CA
- Berkeley County, SC
- Davis County, UT
- Dekalb County, GA
- Grants Pass, OR
- Orleans County, NY
- Richland County, SC
- St. Louis County, MN
- Warrick County, IN
- Waterford, MI
- Woodbury, MN
- Tiverton, RI
- Rochester, MN
- Newcastle, DE
- Lynchburg, VA
- Manchester, CT
- Alexandria, VA
- Bartholomew, IN
- Bayfield County, WI
- Brazos County, TX
- Cass County, ND
- Pleasanton, CA
- Chochise, AZ
- Coon Rapids, MN
- Cook County, IL

# Get your free demo BACtrack breathalyzer

monitoring.bactrack.com  
415-693-9194  
view@bactrack.com

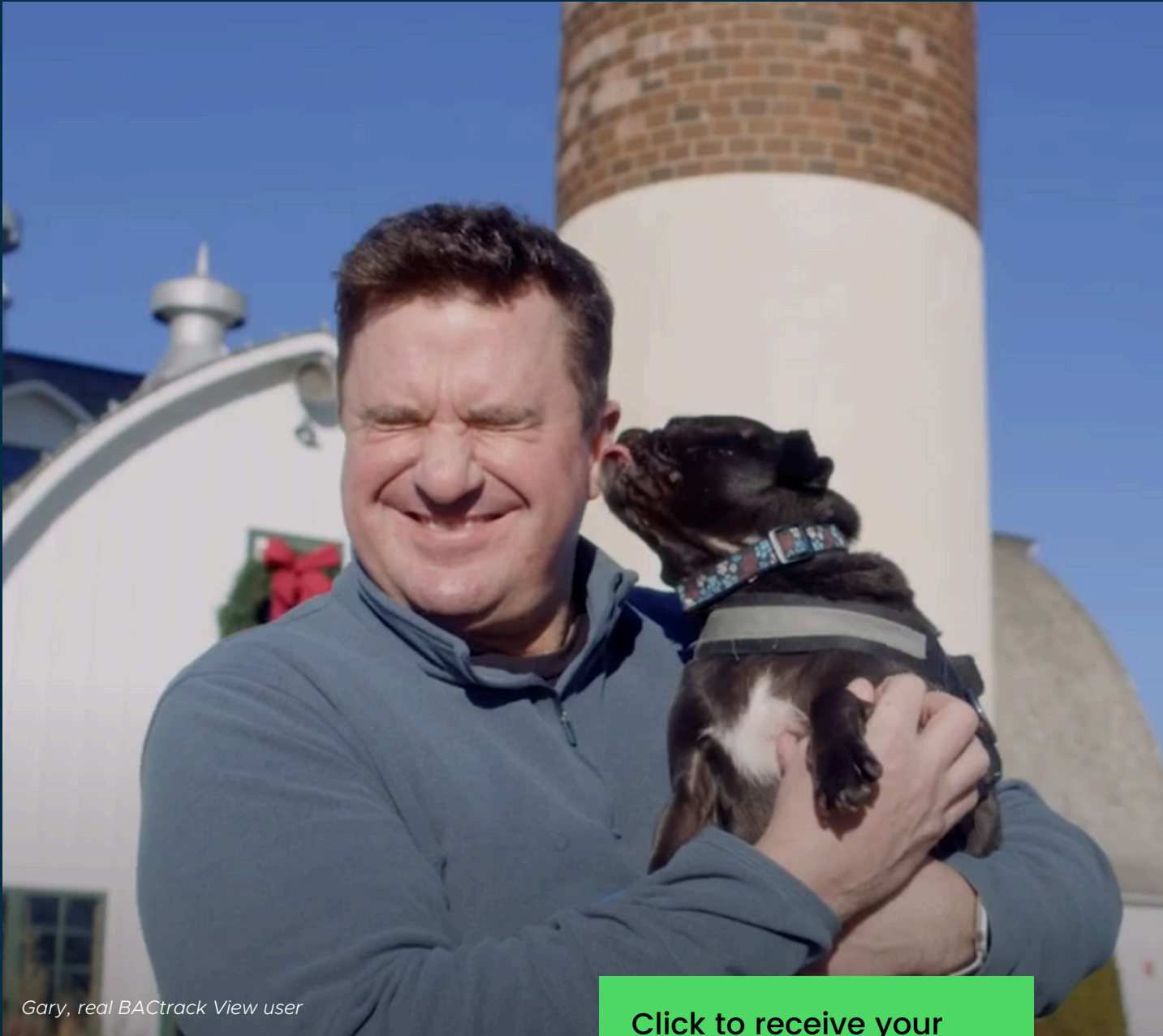

*Gary, real BACtrack View user*

**BAC**track® | VIEW

Click to receive your  
free demo breathalyzer  
(available to Family  
Law Attorneys  
in the US and  
Canada)

